

A natural part of stoma care

Looking after your skin is an essential part of the long-term management of your stoma. But sometimes a simple appliance is not enough. That's why Convatec skin care accessories can serve as an important accompaniment to your daily routine.

Stomahesive® Powder

If you have skin around your stoma that is moist or at risk of breakdown due to moisture, then you should try Convatec Stomahesive® Powder. Why? Because Stomahesive® Powder is specifically designed to protect moist skin around the stoma and encourage healing and restoration by absorbing excess moisture that can result in skin breakdown. And it contains pectin, **an extract of citrus fruit**. Convatec Stomahesive® Powder works by forming a protective barrier against the moisture that seeps from damaged or weeping skin. It is compatible with Convatec appliances and other pouching systems.

See usage instructions overleaf.



USAGE INSTRUCTIONS

1. Clean the peristomal skin and dry thoroughly.
2. Apply a light dusting of powder over the affected areas (deeper areas may require more powder).
3. The powder will form a 'putty-like' substance that acts as a protective film under the wafer and around the stoma.
4. Brush away any excess that can interfere with wafer adhesion and apply a wafer as usual.
5. Discontinue using Stomahesive® Powder once the irritated or weeping area has healed.



Stomahesive® Powder also contains gelatin and carboxymethylcellulose (CMC). See pack insert for full usage instructions.

Availability

Stomahesive® Powder is listed on the Australian Stoma Appliance Scheme and are available through your local ostomy association. Residents of New Zealand should contact the Stomal Therapy Nurse in their local district health board to obtain product.

PRODUCT CODE	DESCRIPTION	QUANTITY	SAS CODE	MONTHLY ALLOWANCE
25510	28gm Bottle	1	3511T	2



Find support on every step of your ostomy journey with the me+® program. To find out more or to request samples, please contact our customer care team:

Australia Call **1800 335 276** or email connection.au@convatec.com
New Zealand Call **0800 225 4309** or email connection.nz@convatec.com

ALWAYS FOLLOW THE DIRECTIONS FOR USE

®/™ indicates trademark of Convatec Inc. ©2024 Convatec Inc.
AP-66956-AUS-ENG-v1 (v1.3) 0239.2 November 2024

