



PRE-CUT | CUT-TO-FIT | MOULDABLE |  
CONVEX | ACCORDION


## Stoma care **Application Guide**

Convatec Natura® and Natura®+  
Two-piece Solution





# Natura® & Natura®+ APPLICATION GUIDE

## CHOOSE BASEPLATE


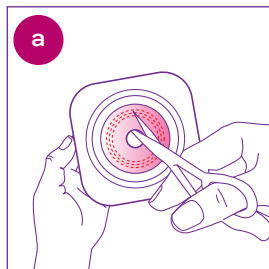


**a. Cut-To-Fit** Trace the shape of the stoma on to the baseplate using your stoma cutting guide. Cut the baseplate just outside the tracing, taking care not to cut the pouch film. Smooth cut opening with finger.

**b. Pre-Cut** Select pouch with pre-cut hole slightly larger than your stoma.

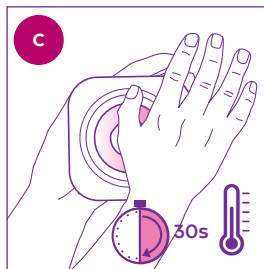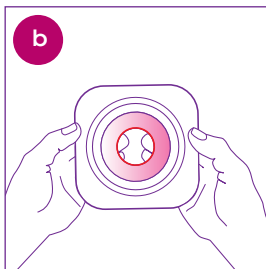


**c. Mouldable** Warm the mouldable baseplate between your hands for 30 seconds. Roll and shape the Mouldable Technology™ to the size and shape of your stoma. **Do not stretch, pull or cut.**

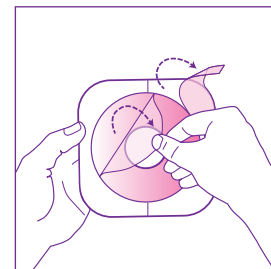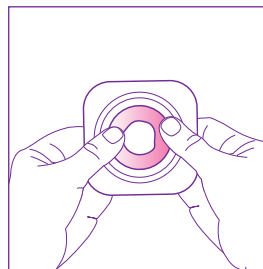


Remove release liner.


## APPLYING THE POUCH

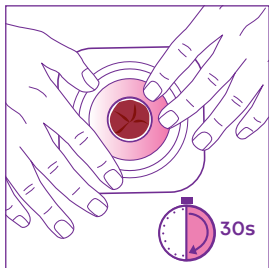


Press the baseplate into place. Smooth and hold for 30 seconds.

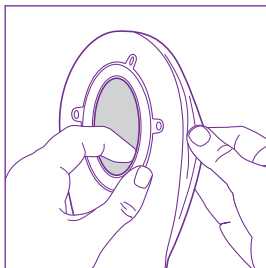

Separate pouch films to allow air to enter.

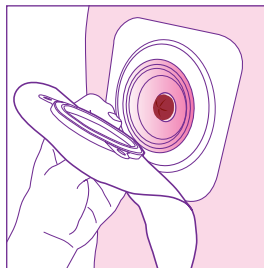


Start at the bottom and apply pressure around the coupling until the pouch and baseplate snap together.

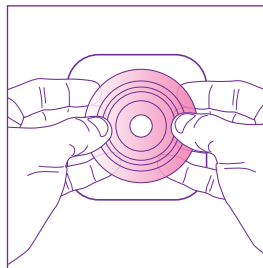


**Attaching to accordion baseplate** **PULL UP** Accordion baseplate and hold with thumbs on top and fingers underneath. Align coupling of pouch and baseplate. Start at the bottom and apply pressure around the coupling until the pouch and baseplate snap together. **COLLAPSE** the Accordion baseplate in the **DOWN** position for a flat profile.




## CLOSING YOUR DRAINABLE POUCH (choose your tail closure)

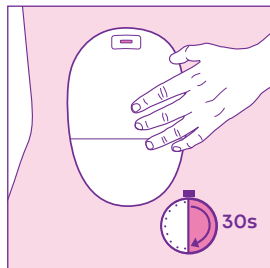

Smooth and hold in place for 30 seconds. **Always double check the coupling is completely attached, if not repeat the steps.**

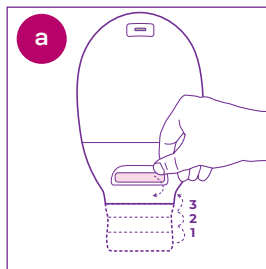


**a. Invisiclose® Closure** Fold tail closure with outlet strip until interlocking closures overlap. Press interlocking closures together across the width of the tail and squeeze until you feel them close. **Lock-it Pocket™** Turn pocket inside-out to conceal the tail closure.

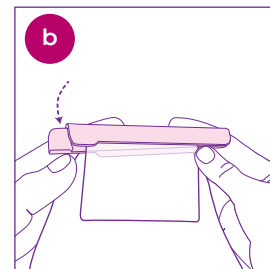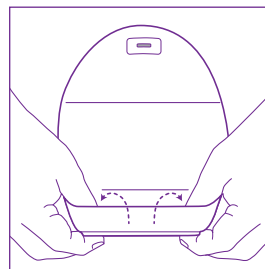
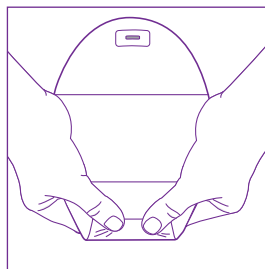


**b. Clipped Closure** Once tail is folded clip in to place.


## EMPTYING YOUR DRAINABLE POUCH (choose your tail closure)

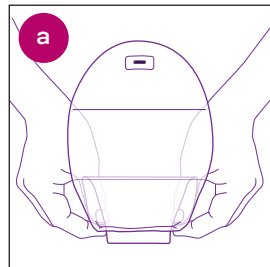

**a. Invisiclose® Closure** Take out the tail from Lock-it pocket™. Unfold pouch closure and open the tail by squeezing in the outlet strips; drain pouch.

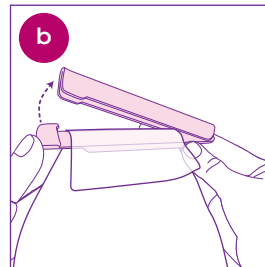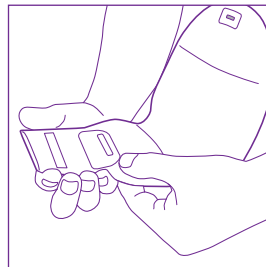

**b. Clipped Closure** Remove clip to empty pouch.

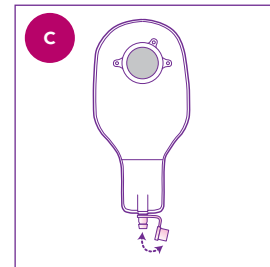

**c. Remove cap of tap to drain.**





## FILTER



Cover the filter with a filter sticker for any water activity and remove after activity.

## HIGH OUTPUT POUCH





## UROSTOMY POUCH - OPEN, CLOSE AND CONNECTING TO NIGHT BAG (follow instructions for your pouch tap. Each tap type has its own connector.)

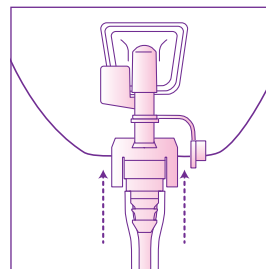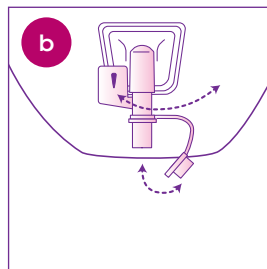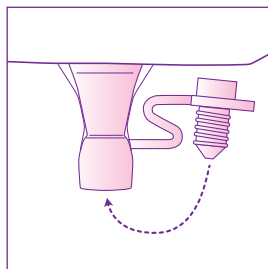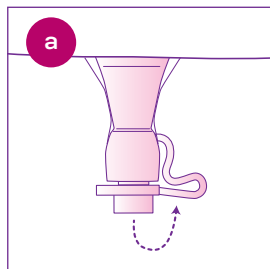

**a. Natura®+ Soft Tap**  
Open and close.

**b. Accuseal™ Tap**  
Open and close.

Connect Accuseal™ connector.

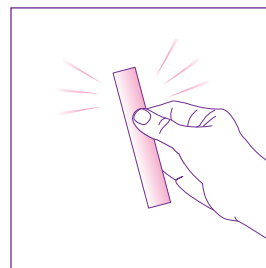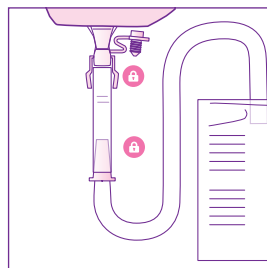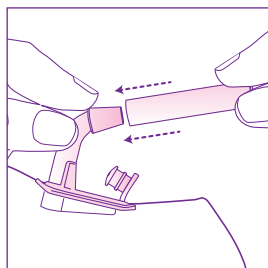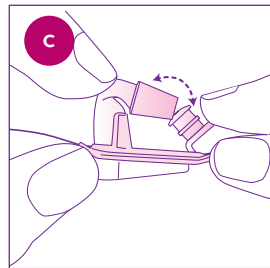


**c. Fold-up Tap**  
Place tap in or out of tap holder to open and close.

Connect Fold-up Tap connector.

**Connecting your night bag**  
Once your pouch tap is open, attach the connector to the tap and on the other end connect your night bag to the connector. Check all connection points are secure.

**Retain your connector** each time you dispose of your night bag.

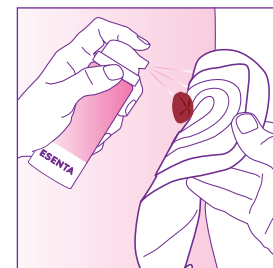

For gentle removal use Esenta™ wipe or spray around edge of the baseplate and wait a few seconds. Gently peel baseplate away from skin.

## REMOVAL

For more information call UK **0800 282 254** ROI **1800 721 721**

# Accessories

## Support Products

Accessories - designed for easy daily use with one-piece and two-piece stoma solutions.



### Pouch Removal



#### ESENTA™ Sting Free Adhesive Remover Spray and Wipes

ESENTA™ Sting-free adhesive spray and wipes quickly remove all adhesive residue and help to reduce painful skin stripping and skin damage. For fast and easy adhesive release feeling silky and smooth.



### Skin Preparation



#### ESENTA™ Sting Free Skin Barrier Spray and Wipes

ESENTA™ skin barrier creates a breathable barrier on the skin, reducing the risk of skin damage and irritation caused by body wastes, enzymes, and adhesives. Safe and effective even with frequent use.



### Added Security



#### Diamonds™ Gelling and Odour Control Sachets

Super absorbent gelling sachets with unique ActiveOne™ odour control. Prevents leaks, odour and noise. Provides a flatter, discreet pouch.

To order call UK **0800 282 254** ROI **1800 721 721**

# Helpful numbers

To speak to one of our Product Specialists

Call UK **0800 282 254** - ROI **1800 721 721**

To speak to one of our me+™ Advisors

Call UK **0800 467 866** - ROI **1800 721 721**

To speak to our Clinical Nurse Specialist

Call UK **0800 085 2516** - ROI **1800 721 721**

To speak to one of our Amcare™ Agents\*

Call **0800 88 50 50**



\* Not available in Ireland. 1 - Reference: 1. Voegeli D, LBF® "No-Sting" Barrier Wipes: Skin Care Using Advanced Silicone Technology. British Journal of Nursing, 2008; Vol. 17: No. 7.  
©2023 Convatec®/™ all the trademarks are the property of Convatec group companies. AP-030474-MRL-GB OST341v4



Printed on FSC mix recycled paper. FSC forests are managed with consideration for people, wildlife and the environment.