Glide

with
FeelClean™
Technology

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Please read all instructions before using this device.

English
en

Hydrophilic Intermittent Urinary Catheter

Indications

- Intermittent catheters are indicated for routine transient drainage of the bladder. The catheter is inserted through the urethra.
- Contraindications**
- A healthcare professional should assess the need for intermittent catheterization and its contraindications.
- Do not use in conjunction with irrigation or instillation.
- Cautions, precautions & observations**
- Check the catheter's integrity prior to insertion and that it is the correct size as indicated on the packaging.
- Select a replacement catheter, if required.
- Do not use if package has been previously opened or damaged.
- If required, connect a urine bag to the funnel end of the catheter before use.
- Do not touch the catheter tip and use the no-touch sleeve to minimize contact with the rest of the catheter.
- Squeeze the water sachet to burst it inside the pack and cover the catheter tube with water, then use within 2 minutes.
- Always slide the catheter slowly and smoothly when inserting and removing. Never force the catheter.
- When removing the catheter, stop if more urine starts to flow. Continue again once it stops.
- The blue sticky dot on the back of the pack can be used to attach the pack to a surface for storage while using the catheter.
- If there is no urine flow or other complications occur (e.g. pain, difficult insertion or removal, bleeding, signs of urinary tract infection or discharge) please consult a healthcare professional.
- After incorrect insertion, catheters must be discarded and not reused. A new catheter should be selected.
- Used catheters should always be discarded.

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