



Starting out towards confident catheterization

We know it can be difficult to find out you need to start using an intermittent catheter.

This introduction is here to help guide you and give more information about intermittent catheterization as you start your journey.

Using a catheter is a life change but it doesn't have to change how you live your life

When deciding how intermittent catheterization will work for you, it might help to keep in mind the things in your life most important to you.

Your health and well-being are very important in your decisions, but so are things like your independence, personal relationships (including intimate ones), hobbies, and perhaps even that dream vacation you've always wanted to take.

Intermittent catheterization should not get in the way of any of these things. It will become easier with practice and can support your lifestyle and help you live the life you want to lead.

Ready? Let's begin together

Why might you need to use an intermittent catheter?

People sometimes have difficulty passing urine freely, here are a few reasons why some experience problems.1-4


• An enlarged prostate in some men can get in the way of the free-flow1 The nerves don't -Rectum signal for bladder Bladder muscles to push Prostate urine out. Sphincter muscle Anus Damage to the spinal Urethra cord can disrupt the communication between the brain and various parts of Catheter the body, including the bladder. Bladder The muscle surrounding the bladder itself does not work properly.1-2 Rectum Catheter Anus Vagina Urethra

There maybe something blocking the urine.

• The sphincter muscle may not open to release urine¹⁻²

How catheterization works. Helping to drain urine from your bladder

In intermittent catheterization, you put a catheter - a hollow tube - into the urethra (which is where urine travels from the bladder to leave the body).

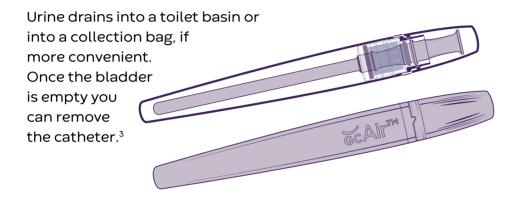

Why is keeping a full bladder for too long a health concern?

Your bladder needs emptying 4 to 6 times a day.3

Other than the discomfort it can cause, urine accumulating in your bladder can

- damage the wall of your bladder²
- allow sediment in urine to build up and form bladder stones⁵⁻⁷
- increase your risk of urinary tract infections (UTIs)²
- lead to uncontrolled urine leakages²

Why might intermittent catheterization be for you?

If difficulty emptying the bladder has become a health concern, intermittent catheterization is considered the 'gold-standard' treatment option.^{8,9}

Unlike with other catheterization options, such as indwelling catheters, the catheter doesn't need to be attached to you all day and night; instead, you use the catheter only when you need to urinate (that's why it's called "intermittent" catheterization).

Regularly emptying your bladder throughout the day with intermittent catheterization can help you stay in control of your bladder health and minimize the impact on your life.

What is indwelling catheterization?

With indwelling catheters (sometimes called a 'Foley') a healthcare professional inserts the catheter through your urethra into your bladder. A small balloon holds it in place inside the bladder, where it is kept for as long as 1 to 3 months.¹⁰

Compared with indwelling catheterization, intermittent catheterization can^{2,11}

- reduce your risk of UTIs
- help protect your bladder
- improve your independence and quality of life by letting you move freely between catheterizations
- reduce barriers to intimacy that other catheterization methods might pose (because you don't need this type of catheter in all the time)

Before deciding if intermittent catheterization is for you

As with any medical intervention, it's important to understand potential risks. The good news is that the risks can be reduced with good training, good technique, and by finding a catheter that works for you.



UTIS

Although the risk of infections is generally lower than with indwelling catheters, UTIs can still happen if bacteria is able to pass up your urethra and into your bladder. 12



Urethral damage

Repeated irritation (often caused by a catheter not being well lubricated or if more training for good technique is needed) can cause scar tissue to build up and make the urethra narrower. You might hear this called 'urethral strictures'.^{2,13}



Bladder stones

This can happen if sediment in your bladder is insufficiently drained and allowed to build up over time or in some cases, when a pubic hair is pushed into your bladder.^{1,5-7}



Urethral irritation

Using a catheter can irritate the urethral lining, which is a more common occurrence when people are first learning how to use a catheter.^{2,13}



False passage

It is possible that the lining of the urethra gets punctured during catheterization, often due to the excessive use of force. If someone gets a 'false passage', they'll need to use an indwelling catheter for several weeks while the urethra lining heals.¹

There's no need to worry about the specific details for now






Selecting a catheter for your lifestyle

Catheters come in different shapes, sizes, and styles to suit different people.^{1,3,14}

Because the urethra is a barrier against germs that can cause infections, it's important to make sure your catheter is well lubricated to avoid discomfort or irritation.²

Not all catheters are the same

Catheters differ in

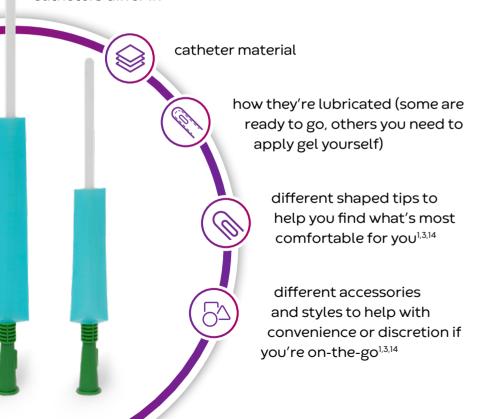

Get support

It's important to talk with your healthcare professional about how you're feeling and openly discuss any concerns.

Convatec me+ Continence Care support is here for you. The program consists of 'Your Personalized Video Guide', educational tools and resources and 10 specially created emotional well-being modules to guide and support you on your catheterization journey.

Helpful resources

Find out more about intermittent catheterization (sometimes called IC) including essential knowledge of anatomy and IC at convatec.com.

- Intermittent catheters: what they are and how to know which one is right for you
- The IC technique: what it is, how it is learned, and how important each step is
- Living with IC: complications to watch out for and how to prevent them
- A confident journey: useful advice and tips, from us and from IC users, and common questions answered



Notes

Scan for additional resources and access to Convatec me+ Continence Care support or visit gr.convatec.com/cc-meplus

REFERENCES: 1. Vahr S, Cobussen-Boekhorst H, Eikenboom J, et al. Urethral intermittent in adults. Published online 2013, 2, Newman D, Willson M. Review of Intermittent Catheterization and Current Best Practices Urol Nurs. 2011;31:12-28, 48; quiz 29. doi:10.7257/1053-816X.2012.31.1.12. 3. Cutting D, Laura Robbs, Laureen Sommerey, Nicholas Joachimides. Clean Intermittent Urethral Catheterization in Adults Canadian Best Practice Recommendations for Nurses, Published online April 2020, https:// unc.org/wp-content/uploads/2020/05/Clean-Intermittent-Urethral-Catheterization-Adults-for-Nurses-BPR-May2020-1.pdf. 4. Davis C, Rantell A. Selecting an intermittent self-catheter: Key considerations. Br J Nurs. 2018;27:S11-S16. doi:10.12968/bjon.2018.27.Sup15.S1. 5. Feneley RCL, Hopley IB, Wells PNT. Urinary catheters: history, current status, adverse events and research agenda. J Med Eng Technol. 2015;39(8):459-470. doi:10.3109/03091 902.2015.1085600, 6. Fogazzi GB, Crystalluria: a neglected aspect of urinary sediment analysis, Nephrol Dial Transplant Off Publ Eur Dial Transpl Assoc - Eur Ren Assoc. 1996;11(2):379-387. doi:10.1093/oxfordjournals.ndt.a027276. 7. Lila ASA, Rajab AAH, Abdallah MH, et al. Biofilm Lifestyle in Recurrent Urinary Tract Infections. Life Basel Switz. 2023;13(1):148. doi:10.3390/ life13010148. 8. RCN Catheter Care Guidelines 2021. NHSGGC. Published February 29, 2024. Accessed July 11, 2024. https://www.nhsggc.scot/ downloads/rcn-catheter-care-guidelines-2021/. 9. Dorsher PT, McIntosh PM. Neurogenic Bladder. Adv Urol. 2012;2012(1):816274. doi:10.1155/2012/816274. 10. Types of urinary catheters. nhs.uk. Published October 23, 2017. Accessed July 11, 2024. https://www.nhs.uk/conditions/urinary-catheters/types/. 11. Rantell A. Intermittent self-catheterisation in women, doi:10.7748/ ns2012.06.26.42.61.c9164.12. Flores-Mireles AL, Walker JN, Caparon M, Hultgren SJ. Urinary tract infections: epidemiology, mechanisms of infection and treatment options. Nat Rev Microbiol. 2015;13(5):269-284. doi:10.1038/nrmicro3432.13. Additions from KOL meeting.14. Newman DK. Teaching Tool: Intermittent Self-Catheterization Patient Education Checklist. Urol Nurs. 2021;41(2):97. doi:10.7257/1053-816X.2021.41.2.97.

 $\ 0.2024$ Convatec. "\" indicate trademarks of the Convatec group of companies. Convatec me+ is a trademark of Convatec Ltd. AP-70154-GBL-ENG-v1.

